

## **ENDO TIPS - Bula**

### **PORTUGUÊS**

DEFINIÇÃO: Pontas para aspiração. COMPOSIÇÃO: Polipropileno.

ADVERTÊNCIAS/PRECAUÇÕES: Descartar após o uso. INSTRUÇÃO DE USO: 1. Desinfecte as pontas através da imersão por 30 minutos em Solução de Hipoclorito de Sódio a 1% ou Solução Álcool Etílico a 70%; 2. Lave em água corrente para remover resíduos seguido de secagem com ar comprimido; 3. Adapte as pontas às cânulas para aspiração. OBS: Se necessário, faça a curvatura das pontas através de pressão na direção e inclinação desejada.

### **ESPAÑOL**

DEFINICIÓN: Puntas para aspiración. COMPOSICIÓN: Polipropileno. ADVERTENCIA/PRECAUCIONES: Desechables después de su uso. INSTRUCCIÓN DE USO: 1. Desinfecte las puntas sumergiéndolas en una solución de hipoclorito de sodio al 1% o en alcohol etílico al 70%; 2. Lave en agua corriente para remover residuos seguido de secado con aire comprimido; 3. Adapte las puntas a las cânulas para aspiración. OBS.: Si es necesario, haga la curvatura de las puntas por medio de presión en la dirección e inclinación deseada.

### **ENGLISH**

DEFINITION: Aspirating tips. COMPOSITION: Polypropylene. WARNING/PRECAUTIONS: Disposable after use. INSTRUCTIONS FOR USE: 1. Disinfect the tips by immersion in 1% sodium hypochlorite or 70% ethanol for 30 minutes; 2. Wash with water to remove debris and dry with air; 3. Attach tips to cannulas for suction. OBS.: If necessary, the tips may be bended by curving them towards the required direction and inclination.