SERINGA TRÍPLICE KIDS

JERINGA TRIPLE KIDS | KIDS AIR/WATER SYRINGE

PORTUGUÊS

GENERALIDADES

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ENGLISH

GENERAL INFORMATION

Aiming to reduce dental treatment anxiety, the KIDS TRIPLE SYRINGE was developed to be used by the dentist in a playful manner with the patient.

DEFINITION

Triple syringe for Pediatric Dentistry.

INDICATIONS

For dental procedures that require air/water spray.

COMPOSITION

- Triple syringe: stainless steel and aluminum.
- Adhesive stickers: thermopolymerized epoxy resin.

TECHNIQUE FOR USE

1. Remove the syringe aluminum handle by turning it counterclockwise;
2. Pass the air and water tubes of the dental equipment through the handle and connect them to the air and water terminals on the syringe head. Be careful to connect them properly: the water terminal (smaller) is at the left and the air terminal (larger) is at the right (see the illustration);
3. Reattach the syringe handle to the head;
4. For greater infection control, replace the conventional metal tip for a disposable Triple Tip.

Ⓢ **IMPORTANT:** adjust the air/water pressure following the instructions of your dental equipment manufacturer.

USING THE ADHESIVE STICKERS

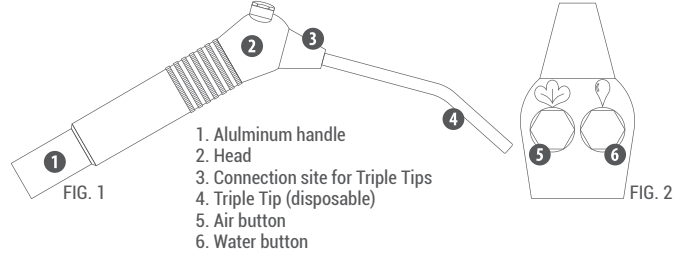
The colorful stickers available with the syringe may dislodge depending on use frequency and time. It is recommended to replace them every 3 months.

Replacing the sticker: remove with finger pressure, clean the application site with alcohol and roughen it lightly with a low speed diamond bur; place a new sticker and press it firmly with finger pressure.

MAINTENANCE

Provide proper care and maintenance to avoid air leakage, water drips and malfunctioning of the buttons. Repairs should be performed by trained personnel.

DESCRIPTION OF COMPONENTS

PROCEDURES IN CASE OF DEFECTS

- Before calling for an evaluation or repair of your syringe, have it available along with its model, batch number and fabrication date.
- Contact Angelus Customer Service – 0800 727 3201, (+55 43) 2101 3200 – out of Brazil or write to sac@angelus.ind.br
- For quicker service, please have the purchase receipt also available.

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ATTENTION: This product must be used according to the instructions described in this manual. The manufacturer is not responsible for failure or damage caused by incorrect handling or use.

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